

# Exo Iris<sup>®</sup>

Version 3.0.2 Mobile



## INTRODUCTION


Exo Iris<sup>®</sup> is a category-redefining 3-in-1 handheld ultrasound device offering a complete medical imaging ecosystem for use, well, anywhere. Powered by our proprietary silicon technology, Iris integrates advanced imaging, intuitive workflow and real-time artificial intelligence (AI) to deliver immediate answers at the point of care.

## KNOWN ISSUES

- After ending and then resuming a scan while in Pulse-Wave-Doppler mode, a blank screen may be displayed when scanning is resumed. As a workaround, tap on “unpause”, and live imaging will resume.

# Exo Works®

Version 3.0.2 Web and Mobile

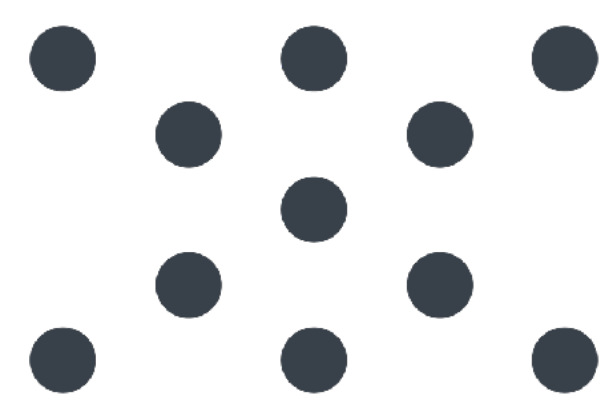

## INTRODUCTION

Exo Works® is an intuitive POCUS workflow software designed around clinicians. Exo Works® helps physicians document and bill exams quickly, and easily manage QA from anywhere.

## WHAT'S NEW

- During SmartLink installation, users can now choose between Microsoft SQL Express and a remote Microsoft SQL Server.

## WHAT'S FIXED

### SmartLink:

- When querying an ADT worklist from an ultrasound modality, SmartLink now sends the patient name in the correct sequence: Last Name, First Name, Middle Name, Prefix, Suffix.
- Patient data from the EHR is now transferring correctly to Exo Works, regardless of whether the patient's middle name is present.

## KNOWN ISSUES

### Mobile App:

- When configuring PACS connections in the Essential license tier, multiple PACS can be set as defaults. As a workaround, uncheck any PACS connections that should not be set as the default.

## KNOWN ISSUES

### Web App:

- When a user downloads a CSV file, the file name is displayed in UTC instead of the user's local time zone.
- In Exo Pulse, when a Customer Admin is reassigned, the new Customer Admin does not receive a welcome email. As a workaround, add the new Customer Admin as a Specialty Admin in Exo Works before updating the Customer Admin in Exo Pulse.
- In an *Order-Based workflow*, if an order is **not** assigned to an exam, no error message appears in the card or list views. To see the error message, open the the exam.

### SmartLink:

- Inactive specialties are synced to SmartLink without any indication that they are inactive. To check on active/inactive status, review the Exo Works Admin page.

To learn more about Exo Iris® and Exo Works®, please visit [exo.inc](http://exo.inc) or contact us at [exosupport@exo.inc](mailto:exosupport@exo.inc).

Exo Imaging, Inc. ("Exo") has taken care to ensure the accuracy of this document. However, Exo® assumes no liability for errors or omissions and reserves the right to make changes without further notice to any products herein to improve reliability, function, or design. Performance varies depending on system configuration. No product or component can be absolutely secure.

This document contains proprietary information, which is protected by copyright. No license (express or implied, by estoppel or otherwise) to any intellectual property rights is granted by this document. Unauthorized copying of this document, in addition to infringing copyright, might reduce the ability of Exo to provide accurate and current information to users. No part of this document may be reproduced, modified, distributed or otherwise used without prior written permission.

Exo®, Exo Iris®, Exo Works®, and the Exo dots logo are trademarks of Exo Imaging, Inc. in the U.S. and/or other countries.

Copyright 2026, Exo Imaging, Inc.

DOC-0313 Rev. A

Exo Imaging, Inc. 4201 Burton Drive, Santa Clara, CA 95054 USA

Telephone: 1-833-MEET-EXO (1-833-633-8396)