

Exo Iris[®]

Version 3.1 Mobile



INTRODUCTION

Exo Iris[®] is a category-redefining 3-in-1 handheld ultrasound device offering a complete medical imaging ecosystem for use, well, anywhere. Powered by our proprietary silicon technology, Iris integrates advanced imaging, intuitive workflow and real-time artificial intelligence (AI) to deliver immediate answers at the point of care.

WHAT'S NEW

ScanTutor Enhancement:

- There is new view available in the Cardiac ScanTutor suite: Parasternal Short Axis (PSAX) view to the Cardiac ScanTutor suite.

Imaging Improvements:

- Sharper resolution for linear and abdominal exam types.

Iris Data Analytics:

- Option to export up to 30 days of anonymized Iris activity in a CSV format.

iOS Support

- iOS 18.6 or later is required to use the Exo Iris app.

KNOWN ISSUES

- On iPad, the Login button on the Welcome page becomes unresponsive when rotating the device from portrait to landscape orientation. Exo recommends logging in to the app before rotating the device.

Exo Works®

Version 3.1 Web and Mobile



INTRODUCTION

Exo Works® is an intuitive POCUS workflow software designed around clinicians. Exo Works® helps physicians document and bill exams quickly, and easily manage QA from anywhere.

WHAT'S NEW

- View at-a-glance billing insights for you and your department, and export data as CSV.
- Optional credentialed sign-off for Educational exams—making QA review quick and easy.

WHAT'S FIXED

- When a user downloads a CSV file, the file name is now displayed in the user's current local time zone.
- In Pulse, when a Customer Admin is reassigned, the new Customer Admin now receives a welcome email.

KNOWN ISSUES

Mobile App:

- When configuring PACS connections in the Essential license tier, multiple PACS may be set as defaults. As a workaround, uncheck all PACS connection(s) that should not be set as the default.
- For Connect and Enterprise license tiers, once a worklist has been configured for a specialty, it no longer appears on the Settings page. However, the connection remains fully functional, and worklist orders continue to be available to users within that specialty for assignment to the exam.

KNOWN ISSUES - CONTINUED

Web App:

- For an *Order-Based Workflow*, if an order is not assigned to an exam, an error message indicating that an order is missing is not shown in the card or list views. The error is shown when the user opens the exam.
- When a user is created in Pulse, the UI may not update immediately. As a workaround, refresh the page to view the newly created user.

SmartLink:

- Inactive specialties are synced to Smartlink with no indication that they are inactive. As a workaround, the active/inactive status is displayed on the Exo Works Admin page.

To learn more about Exo Iris® and Exo Works®, please visit exo.inc or contact us at exosupport@exo.inc.

Exo Imaging, Inc. ("Exo") has taken care to ensure the accuracy of this document. However, Exo® assumes no liability for errors or omissions and reserves the right to make changes without further notice to any products herein to improve reliability, function, or design. Performance varies depending on system configuration. No product or component can be absolutely secure.

This document contains proprietary information, which is protected by copyright. No license (express or implied, by estoppel or otherwise) to any intellectual property rights is granted by this document. Unauthorized copying of this document, in addition to infringing copyright, might reduce the ability of Exo to provide accurate and current information to users. No part of this document may be reproduced, modified, distributed or otherwise used without prior written permission.

Exo®, Exo Iris®, Exo Works®, and the Exo dots logo are trademarks of Exo Imaging, Inc. in the U.S. and/or other countries.

Copyright 2026, Exo Imaging, Inc.

DOC-0314 Rev. A

Exo Imaging, Inc. 4201 Burton Drive, Santa Clara, CA 95054 USA

Telephone: 1-833-MEET-EXO (1-833-633-8396)